



Aquilo Sports

Knee/Elbow Wrap

Operator's Manual



Rev. 1

KNEE

Required: Mode 2 (M2) // Recommended: 50mmHg

Instructions:

1. Unzip the wrap and lay it out flat.
2. Connect air and water hoses from the Control Unit to the wrap.
3. Place the knee inside the center of the wrap.
4. Zip the wrap closed around the knee.
5. **Program the compression system on the CCT1500 device to Mode 2.**
6. Start treatment session. (See Aquilo Sports CCT1500 Operator's Manual for instructions)

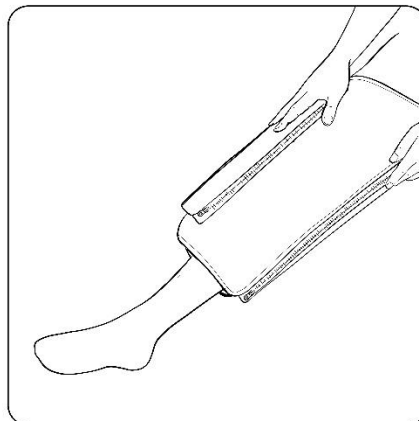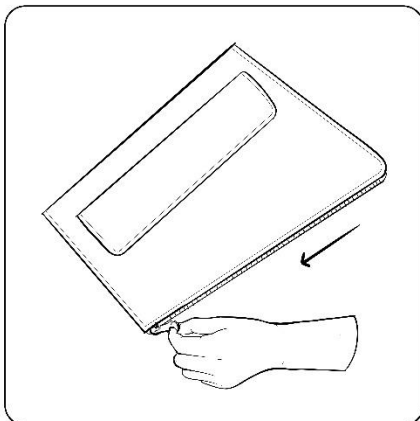



Aquilo Sports

Knee/Elbow Wrap

Operator's Manual

Rev. 1

ELBOW

Required: Mode 2 (M2) // Recommended: 50mmHg

Instructions:

1. Unzip the wrap and lay it out flat.
2. Connect air and water hoses from the Control Unit to the wrap.
3. Place the elbow inside the center of the wrap.
4. Zip the wrap closed around the elbow.
5. **Program the compression system on the CCT1500 device to Mode 2.**
6. Start treatment session. (See Aquilo Sports CCT1500 Operator's Manual for instructions)